

6" DENTA-SLEEVE

Disinfecting Solution Delivery Sleeves



DENTA-SLEEVE

Unlike traditional 4 x 4 gauze sponges, **Denta-Sleeves** provide complete contact of your disinfectant with dental device surfaces to assure no bare spots. Denta-Sleeves also keep sterilized instruments covered until use. Cover suction controls, saliva ejectors, slow- and high-speed handpieces, intraoral scopes and other sensitive instruments.

100 sleeves per bag. Non-sterile. Denta-Sleeves do not contain cellulose.

CLINICAL INSTRUCTIONS

1. Place one day's worth of Denta-Sleeves into a disinfection solution. Let soak for at least 15 minutes before use.
2. Place a saturated sleeve on a clean instrument ready for disinfection. Denta-Sleeve offers a disinfection process, not a cleaning process.
3. Return the instrument to the delivery system and allow adequate time for disinfection between patients (a minimum of 10 minutes is required, or longer if possible).
4. Discard the sleeve, rinse the item with tap water, and dry. Since some disinfectants/sterilants are irritating to the skin, follow the manufacturer's directions regarding skin exposure.

Note: If autoclaving handpieces, Denta-Sleeve can be used as a cover in the autoclave and remain on through lubrication and replacement to the delivery system, eliminating the plastic autoclave bag. Some handpiece manufacturers do not recommend bagging handpieces due to heat build-up. If used in this way, disinfection solution would not be used on the Denta-Sleeves.

PRECAUTIONS

- Sterilants are toxic. Wear gloves and protective eyewear.
- Do not allow the solution to run down the internal shaft of a slow-speed handpiece. Invert handpiece if necessary.
- Some handpiece manufacturers do not recommend the use of cold sterilants to disinfect handpieces. The decision to use sterilants is the sole responsibility of the dental practitioner.

Manufactured for:



www.indigreen.com