

AZÚR™

Premium Safety Glasses

fig. 1

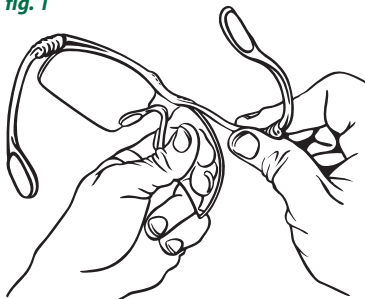


fig. 2



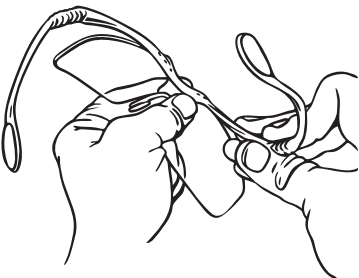


fig. 3

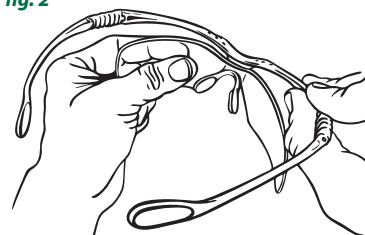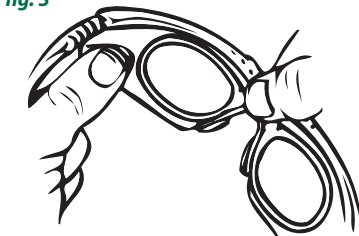

AZÚR SAFETY GLASSES

Thank you for your purchase of Azúr Safety Glasses. Your Azúr glasses were designed especially for health care professionals. We sincerely hope that their advanced featherweight frame, gentle universal fit and unsurpassed all-day comfort make your work more enjoyable.

Your Azúr Glasses may be used with any of the three scratch-resistant polycarbonate one-piece lenses included.

- A clear fog-resistant safety lens
- An orange curing light safety lens
- A tinted lens for patient use

Azúr one-piece lenses are quickly interchangeable by following these instructions:

TO REMOVE LENSES

1. Beginning with either side of the frame, lift the temple and frame up approximately 45° so that the top edge of the lens on that side may be rotated out of its retention slot above the nosebridge (see fig. 1). The plastic frame is highly flexible and will not break when bent in this way.
2. Leave the top edge of the lens resting out of its retention slot while performing the same procedure with the other side of the frame. Once the other top edge is rotated out of its retention slot, the entire lens will pull away from the frame.

TO REPLACE LENSES

1. Begin by resting the V-shaped nose area of the lens into the lower slot of the nosebridge closest to the nose pads (see fig. 2).
2. While pushing the lens forward from behind the nosebridge, lift both sides of the frame up slightly until the lens snaps into the retention slot above the nosebridge. It may be helpful to press one side at a time rather than both sides at once.

TO REMOVE/REPLACE PRESCRIPTION INSERTS

1. Begin by resting the V-shaped nose area of the prescription insert onto the nose pads (see fig. 3).
2. Push the lens forward from behind the nosebridge until the lens prescription insert snaps into the frame.

CARE AND CLEANING

While Azúr Lenses are made of 100% polycarbonate for superior impact-resistance and scratch-resistance, they should be treated with care to provide longer lasting protection. To help protect your eyewear from damage, we recommend:

1. Never lay your glasses lens down on a hard surface or put your eyewear away without its protective case.
2. Always clean the lenses with a soft cloth and lens cleaner intended for plastic lenses. Warm water and lotion soap may also be used. Never dry lenses with a paper towel or any abrasive cloth. Air-drying is best.
3. Wash hands before changing lenses to keep lenses as clean as possible.
4. The plastic temples may be reshaped under hot tap water for a custom fit. Make slight adjustments at first, then wear your glasses for a short period to check for proper fit before over-adjusting. Azúr's lightweight and ergonomic design eliminates the need for a tight fit.
5. Alcohol-based cleaners should not be used on polycarbonate lenses.