

# FACIAL PLANE RELATOR

Midline and Horizontal Plane Transfer Device

Facial Plane Relator records exact position of desired midline, for smiling results!

*Esthetic rehabilitation requires detailed planning and good communication. Making sure that a patient's facial midline and the dental mid-line coincide—and communicating this accurately to the lab—has always been a challenge, until now!*

With the Facial Plane Relator, you can easily record an accurate bite registration and the exact position of a desired midline in relation to a horizontal plane in just 60 seconds! It is the ideal communication tool between you and your lab for determining the cant and incisal edge position of the central incisors.

The technique is simple and straightforward, and may be used with your favorite bite registration material. The "horizontal plane" alignment bar features an arch with 25 fingers for stabilizing it within the registration material, plus a snap-on vertical alignment bar. Place the arch into labially extruded bite registration material and align it with the patient's face before the material hardens. When your lab tech mounts the Facial Plane Relator onto the working model, all of the usual guesswork about your patient's midline and horizontal planes will disappear along with poor midline results!


24 horizontal arches and vertical alignment bar sets per box.

11210 Facial Plane Relator



Horizontal  
and vertical  
alignment  
bars snap  
together!

2006  
**REALITY**  
Four Star Award  
★★★★

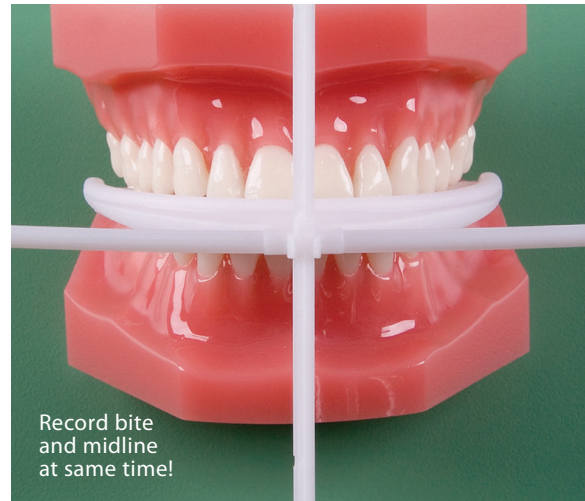

Record bite  
and midline  
at same time!