

# FeatherFlex™

## HVE/SE Adapter

### FeatherFlex Introduction and Installation

With FeatherFlex, you can leave heavy valves and stiff hoses behind! FeatherFlex connects a low volume evacuator to any high volume suction valve by way of a 72" long lightweight hose. In turn, you get more than high volume suction through a low volume aspirator, you also get reduced hand fatigue and improved maneuverability!

To use, insert your low volume evacuator into the low volume end of FeatherFlex with a slight twist, and then insert the high volume end of FeatherFlex into your HVE valve, leaving your HVE valve in its holder.

**Note:** Moistening the tip of the low volume evacuator will make insertion easier.

FeatherFlex works with all standard ¼" diameter low-volume aspirators, including the popular IndiGreen Innovations items shown at left.

FeatherFlex is perfect for use with these aspirators from IndiGreen Innovations:



10111 MirroVac®



12910 Fast Dam. Specify right or left side.




11610 Total Comfort™ with clear tips

11611 Total Comfort™ with blue tips  
Specify white or clear tubing.



FeatherFlex is lightweight, ergonomic and efficient.

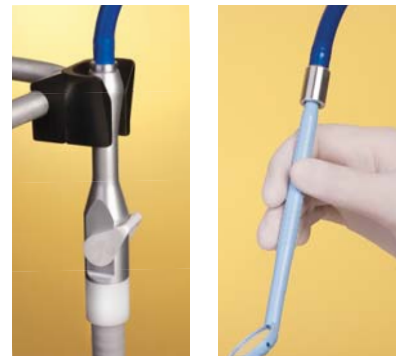

Leave HVE valve in holder!

### FeatherFlex Cleaning and Maintenance

FeatherFlex features a vinyl hose and anodized aluminum fittings. To clean, simply flush along with other evacuator lines. May be surface disinfected.

**Important: DO NOT AUTOCLAVE as heat sterilization may damage the hose.**

The low volume end of FeatherFlex contains an internal O-ring that may have to be replaced occasionally due to wear. This O-ring has an inner diameter of ¼" and an outer diameter of ⅜" (.25" ID x .375" OD). Replacements can be purchased at most major hardware stores or from many dental equipment retailers.

If you have questions or concerns about FeatherFlex, please contact your IndiGreen Innovations dealer or e-mail [customerservice@indigreen.com](mailto:customerservice@indigreen.com).